

Aria® Valve Tool

Part Nr./Ref: CA300-AVT



Read instruction prior to use.



For use with Aria Valve, Aria Valve Plus, Aria Hybrid Valve, Aria Plate Valve and Aria HV Plate Valve

INSTALLATION INSTRUCTIONS

To Remove Valve Cap For Cleaning: Insert top prong of long metal end into space between valve cap and valve body. Gently twist tool to loosen cap from valve. Note: Spring may pop out abruptly once pressure of cap is released.

To Remove Valve From Socket Wall (applies to Aria Valve, Aria Valve Plus and Aria Hybrid Valve):

Once valve cap is popped off using long metal end of tool, remove the spring, rubber diaphragm and center vent spool. This will expose 2 holes on inner face of valve body. Insert the 2 small metal prongs located on one end of valve tool into holes and use tool handle to unscrew valve from socket wall.